

EXAM-VIEWER™ Shortcut Keys

Win/Lvl Preset	Key
Abdomen (CT)	1
Bone (CT)	2
Cerebrum (CT)	3
Liver (CT)	4
Lung (CT)	5
Mediastinum (CT)	6
Pelvis (CT)	7
Posterior fossa (CT)	8
Low contrast (US)	9
Calcium	0

Zoom Factor/Magnification	Key
1x	F2
1.5x	F3
2x	F4
3x	F5
4x	F6
Actual size (will ask for monitor width, mm)	F10

Image layout and display	Key(s)
Display image layout grid	Alt+I
Go to first image in the series	Home
Go to last image in the series	End
Go to next image	↓
Go to previous image	↑
Go to a particular image number	J

Series layout and display	Key(s)
Display series layout grid	Alt+S
Populate next series sequence	PgUp
Populate previous series sequence	PgDn
Populate all series windows (starting with S1; will replace current)	Alt+A

Image manipulation	Key
Apply image changes to current series	A
Edge enhance	B
Fit image in window	F
Flip horizontal	H
Flip vertical	V
Invert (dark/light)	I
Reset image	R
Rotate image, clockwise 90°	C
Rotate image, counter-clockwise 90°	X
Smooth images	N
View animation (ciné)	Q

Localiser	Key(s)
Pop-up	L
Series	Alt+L

Mouse mode	Key
Slice/scroll	S
Win/Lvl	W
Zoom	Z (tap again for zoom box at 3x current magnification)
Pan	P
Marking tool	Key
Measure	M

Cycling Option: Holding down the right mouse button while simultaneously clicking the left mouse button will cycle through the mouse modes and measure tool — from *Slice* to *Win/Lvl* to *Zoom* to *Pan* to the *Measure* tool — without having to click the buttons on the toolbar. Cycling is a continual loop.

Other	Key(s)
Add image to summary	Ctrl+A
Close all exams (exit viewer)	Ctrl+F4
Close current exam	Alt+F4
Copy Image to Clipboard	Ctrl+C
Create CD/DVD	Ctrl+P
Print exam	Ctrl+P
Save All Settings	Ctrl+S
Show DICOM header information	D
Show overlay	O (toggle)
Synch selected series	E (or) Alt+click